

Alpha[®] Classic

AK Liners

Product Highlights

- Unique AK Gel profile adds comfort for transfemoral amputees
- Skin-friendly, mineral-oil based gel
- Available in cushion and locking

Alpha Classic AK Liners are available with an AK Profile in order to meet the needs specific to transfemoral amputees.

Alpha Classic AK Liners with the AK Profile accommodate the often sensitive areas of transfemoral residual limbs with a specific gel style and incorporate skin-friendly, mineral-oil-based Alpha Classic Gel. The liner, offered in cushion and locking, has 9 mm of gel distally, tapering to 3 mm proximally—with a built-in gel pad to protect the often-sensitive femur. This extra gel pad provides a total of 9 mm of gel at the distal lateral aspect of the liner.


The Spirit Fabric used for Cushion Liners is unique in its ability to flex during donning or doffing and may be used with a Velcro[®]-like material for added security. Alpha Classic AK Locking Liners use one-way stretch MAX Fabric that limits vertical pistoning movements around the complete circumference of the liner.



✔ Transfemoral

L-Code Suggestion: L5679 (cushion) L5973 (locking)

Liner Sizing:

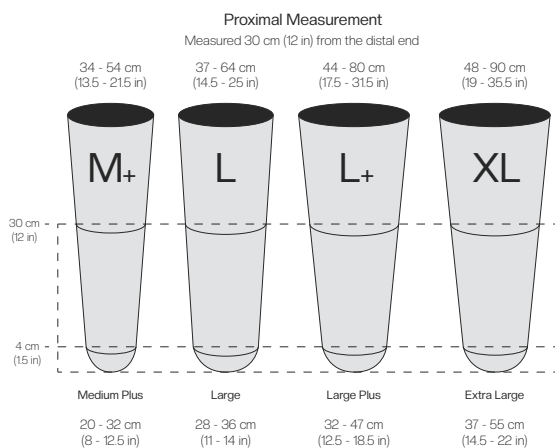

Part Numbers:

Size	Cushion (Spirit Fabric) Item #	Locking (MAX Fabric) Item #	Locking (Select Fabric) Item #
Medium Plus	AKC-2634-E	AKL-2634-X	A442-1494
Large	AKC-2636-E	AKL-2636-X	A442-1496
Large Plus	AKC-2638-E	AKL-2638-X	A442-1498 (Accordion umbrella)
Extra Large	AKC-2637-E	AKL-2637-X (standard umbrella) AKL-2631-X (large umbrella)	A442-1497